

**DENTAL
OFFICE
STEPS**

1st APPOINTMENT

- Take final impression.
- Establish V-D and take bite registration.
(If patient does not have posterior teeth, additional appointment will be necessary to take bite registration with bite blocks.)
- Opposing model or impression.

**SEND TO
PRODENCO**

- Final impression.
- Bite registration.
- Opposing model.
- RX

**PRODENCO
STEPS**

- Create a new model.
- Articulate to the bite.
- Digital or physical extractions.
- Create a digital set up for finish.

TIPS

- Take an impression to capture borders and vestibules.
- Ensure landmarks are present in the impression/model.
- Pour alginate impression immediately.

**2nd APPOINTMENT
FINISH**

- Seat the denture.
- Verify bite, vertical dimension, esthetic, phonetics.
- Adjust denture as needed.



***Call us today
with questions
about this great
new product!***

PRODENCO.COM

611 Omaha St • Sioux City, Iowa
712.252.4034 • 800.831.0936

2218 West 2nd St • Grand Island, NE
308.282.0235 • 800.289.3525

407 N. 203 St • Elkhorn, NE
402.551.5954 • 800.747.5441